Nuevo Formato/ New Format

U-CRP-CONTROL

CRP/PCR-Ultra Control Turbidimetry / Turbidimetria

Quantitative determination of CRP-ultra

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IVD
Conservar a 2 - 8°C

PRODUCT CHARACTERISTICS

The control is a human serum containing human CRP used to evaluate the precision and accuracy of CRP-ultra determinations in human serum by turbidimetry method.

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REAGENTS

U-CRP Control	Human serum. With a low concentration of CRP. Preservative. The CRP concentration is stated on the vial label.
---------------	--

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PRECAUTIONS

Components from human origin have been tested and found negative for the presence of HBsAg, HCV and antibody to HIV (1/2). However handle cautiously as potentially infectious.

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CALIBRATION

The concentration value of the control has been standardized against the Reference Material ERM-DA472/IFCC

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Listo para su uso.

STORAGE AND STABILITY

The control is stable until the expiration date on the label when stored tightly closed at 2-8°C and contaminations are prevented during their use. Do not use reagents over the expiration date.

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IVD
Conservar a 2 - 8°C

PRODUCT CHARACTERISTICS

The control is a human serum containing human CRP used to evaluate the precision and accuracy of CRP-ultra determinations in human serum by turbidimetry method

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